

PRODUCT INFORMATION



EQUIPMENT MAINTENANCE

ERIHYD TERMO

DISINFECTANT



Range of use

Erihyd Termo is intended as an disinfectant for automated washers of endoscopes, which use 20 % glutaraldehyde.

Features

Erihyd Termo disinfectant has a microbicidally effect with wide spctrum. Erihyd Termo exterminates effectively gram+ and gram- and TB bacteria. It is effective also in virus (including HBV and HIV), fungi and spores. pH of the Erihyd Termo concentrate is 3.7. Microbicidally the solution is then at its best and safe disinfection of endoscopes is reached in washers and disinfectors.

Product declaration

| | | |
|------------------|----------|---------------------|
| Glutaraldehyde | < 20 % | microbicide |
| Ethylene Alcohol | 5 - 15 % | stabilization agent |

pH 3.7

Instructions for use

Connect the Erihyd Termo drum with the automatic dosing system of the machine. The endoscope will be connected to the washing and disinfecting cleaner according to the machine manufacturer's instructions. Before storage It is recommended to dry the endoscopes with compressed air. Before using Erihyd Termo make sure that the the product can be used in the type of disinfector washer of endoscopes you have.

Notice

Corrosive. Wear suitable protective clothing and take care of good ventilation. Use biocides safely. Always read the label and product information before use.

Package size / Product code

| | | |
|------------------|------|-----------------------|
| 3 x 5 l canister | 8140 | EAN 64 1260 00 8140 4 |
|------------------|------|-----------------------|

Storage

Warm storage 3 years.

Environmental impact

Packages are made of PE-HD plastic, which is environmentally friendly, recyclable and burnable material. The product musn't let go straght into the environment.

Country of origin

Finland



1/09