


PRODUCT DATA SHEET

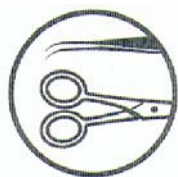

INSTRUMENT DISINFECTION

ERISAN HYDRAGEL

PROTECTIVE GEL FOR INSTRUMENTS

Uses

Erisan HydraGel is a pretreatment product for contaminated instruments that are not washed immediately after use. The product prevents the dirt from drying on the instrument surfaces and the microbes from growing.

Properties

Erisan HydraGel is sprayed on the contaminated instruments. The gel-like spray forms a microbe-proof film on the instrument surface. It prevents the dirt from drying on the instruments and stops microbes from growing and spreading. The effect lasts for 5 days. Erisan HydraGel is water-soluble and easy to rinse out of the instrument surfaces. Suitable for commonly used materials; not suitable for copper or brass. The bacteriostatic effect of the product has been tested at Department of Public Health of University of Helsinki.

Ingredients

Fatty acid alkanol amino salts 5-15%	gelating substance
Glycols 5-15%	drying prevention
Non-ionic tensides < 5%	active detergent
Benzyl hemiformal < 0.1	preservative

Instructions for use

Spray Erisan HydraGel protective gel on the instruments in the transport box. Wet the instruments carefully with the gel. Follow safety instructions for instrument storage and transport. Wash the instruments as usual.

Note

Splashes may make the floor slippery. Clean with water.

Package size/ product code

6 x 750 ml 8222 EAN 6412600082227

Storage

Storage temperature under 28°C. Shelf life 2 years from the manufacturing date.

Ecological information The product is biodegradable. The ingredients are environmentally friendly.

Country of production Finland.



8/07