

PRODUCT INFORMATION



SURFACES AND INSTRUMENTS

ERISAN OXY+

A CLEANING DISINFECTANT

Range of uses

Erisan Oxy+ is a disinfectant for the cleaning and disinfection of medical equipment and instruments. Erisan Oxy+ disinfectant is ideal for disinfecting textiles in low temperatures. It is also suitable for the disinfection of efficient dental suction systems.

Properties

The cleaning power of Erisan Oxy+ is based on hydrogen peroxide and peracetic acid. The product is a granulated powder, the disinfection power of which is activated in a water solution. Disinfection takes place through an oxidation reaction. The product is also suitable for sensitive surfaces and materials. It is not recommended for copper and bronze surfaces. The 50 g single-dose pack makes it easy to mix a solution of the desired strength. The power of Erisan Oxy+ as a disinfectant has been proved by tests: prEN 13727 study of germicidal effect, 13624 study of fungicidal effect, Tbc, HBV, HIV, polio, spores.

Product declaration

Sodium percarbonate >30%
 TAED 5-15%
 Complexing agents > 30%
 Non-ionic tensides < 5%
 Ready-to-use solution pH 7.5

Instructions for use

Prepare the solution in clean and warm (30-40°C) water. Mixing speeds up the dissolution of powder. Once the powder has dissolved, the disinfectant is ready to use. Wipe contaminated surfaces with a folded cloth moistened with the disinfectant solution. Rinse surfaces after disinfection as necessary. Rinse instruments carefully after soaking. Replace the used solution daily.
 Disinfecting wash for heat sensitive materials: for 4-7 kg laundry use 1 single-dose pack of Erisan Oxy+ and half amount of ordinary washing powder. The powders may be mixed together. Use 40°C washing cycle.

Solution	Erisan Oxy/1 l of water	Working time NB	Preservation of clean solution	Uses
2%	20 ml (= 20 g)	-	1 week	disinfection of surfaces
5%	1 single-dose pack	15 min a covered container Polio: 30 min	2 days	disinfection of instruments (soaking)
0,2-0,5 %	1 single-dose pack and c. ½ amount of washing powder in the doser/4-7 kg laundry 40°C washing cycle			for washing and disinfecting heat sensitive clothes and other textiles



Please note

Some foaming occurs in the surface of the solution when dissolving. Prepare the solution in a large enough container. Do not use product if solidified. Classification: Harmful
Use biocides safely. Always read the label and product information before use.

**Package size/
product code**

A box of 50 x 50 g single-dose packs
8116 EAN 641260081169

Storage

Store in temperatures not exceeding 28°C. Shelf life 2 years from date manufactured.

Environmental impacts Biodegradable. Recyclable packaging.

Country of manufacture Finland

