

# PRODUCT INFORMATION



INSTRUMENT MAINTENANCE

## THERMOTON ENDO



DISINFECTANT

### Range of use

Thermoton Endo is a disinfectant for automated washers of endoscopes. It is suitable for most common endoscope reprocessors.

### Features

Thermoton Endo is an effective and quick-acting chemothermical disinfectant based on quaternary ammonium compounds. It has been tested against bacteria (also TB), viruses (HBV, HIV, polio) and fungi. The duration of action is 5 minutes at 55 - 60 °C. Thermoton Endo has a mild fragrance. The product also contains a corrosion inhibitor. pH of ready-to-use solution is 7.5. Product development and manufacturing by Dr. Schumacher GmbH from Germany.

### Product declaration

2-propanol	5 - 15 %
quaternary ammonium compounds	< 5 %
Sodium gluconate	< 5 %

pH 7.5

### Instructions for use

For automated washing of endoscopes use Erinox Termo detergent. Before using Thermoton Endo disinfectant calibrate the reprocessor for a new product. Thermoton Endo dosage is 1 %, duration of action 5 min at 55 - 60 °C. After implementation dosing acts automatically. Annual overhaul has to be done according to manufacturer's instructions.

### Notice

Use biocides safely. Always read the instructions and product information before use.

### Package size / Product code

3 x 5 l canister	8150	EAN 64126000 81503
------------------	------	--------------------

### Storage

Warm storage 3 years.

### Environmental impact

Packages are made of PE-HD plastic, which is environmentally friendly, recyclable and may be disposed of by burning. Do not let the product go straight into the environment.

### Country of origin

Germany. Imported by Farnos Oy



10/08